

NEW
NEEDLE-SAFETY
TRAY SYSTEM



Supported by National Innovation Centre

INSTRUCTIONS



STICK SAFE™

“StickSafe is a simple device that could save the NHS £160m per year”

*Brian Winn, Head of Technology & Product Introduction,
The National Innovation Centre*


www.sticksafe.com

E: info@sticksafe.com


T: 07886 034 601

INSTRUCTIONS

for use with
hypodermic &
vacuum-tube
needles

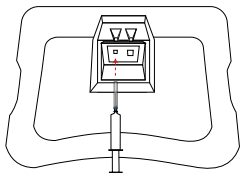


Use small holes for
hypodermic needles

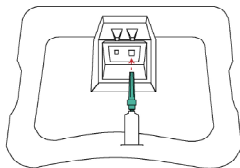


Use larger holes for
vacuum-tube needles


PREPARATION

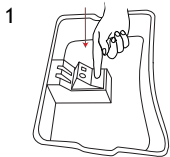

1 Press hinge down before
use

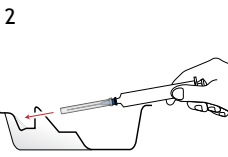


2 Insert syringe with
capped needle

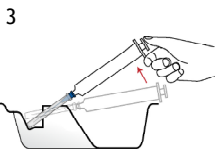


3 Flick up syringe


USE

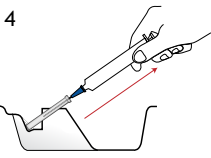

4 Pull out to unclog

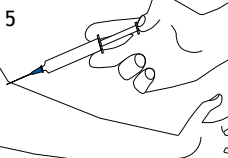


5 Use

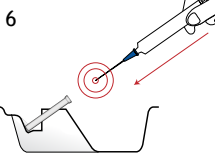


6 Safely recap


REMOVAL

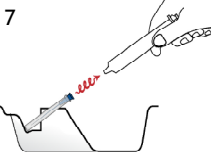

7 Twist of syringe to
denotch

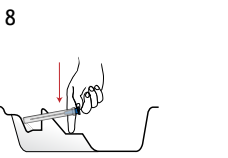

8 Press to release

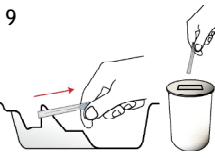

9 Remove capped needle &
dispose